
# DDN Surf

NEUTRAL DISINFECTANT DETERGENT FOR THE  
BIOCLEANING OF FLOORS AND SURFACES

SURFACES TREATMENT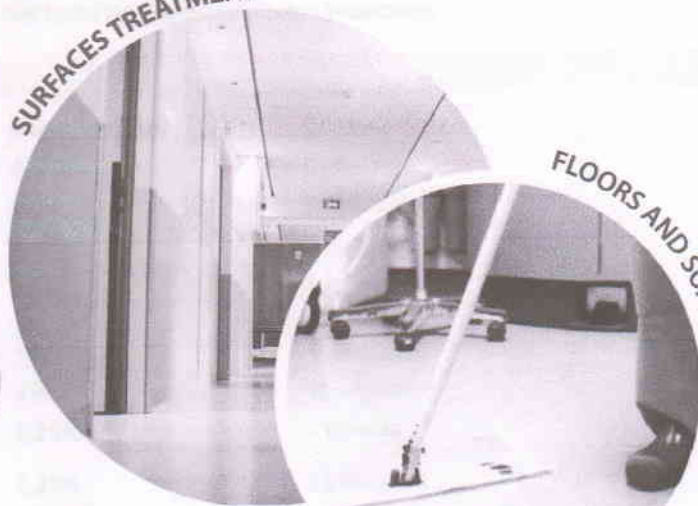

FLOORS AND SOAKING

## PRODUCT ADVANTAGES

- Neutral pH disinfectant detergent
- Active on *Clostridium Difficile*
- Compatible with a wide range of materials
- Mint fragrance
- Biocide registration: TP 02 (inventory N°: 32082)
- Active on enveloped viruses

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**Franklab®**  
Expertise Ultra-Propreté

# DDN Surf

## NEUTRAL DISINFECTANT DETERGENT FOR THE BIOCLEANING OF FLOORS AND SURFACES

### INSTRUCTIONS FOR USE

#### Preparation of the biocleaning solution:

- ☒ Wear suitable personal protective equipment.
- ☒ **For the biocleaning of floors and surfaces in the medical sector:**  
Dilute 20ml of DDN Surf in 8L (0.25%) of water and homogenize the solution
- ☒ **For the biocleaning of floors and surfaces in the food, industrial sectors and community:**  
Dilute 80ml of DDN Surf in 8L (1%) of water and homogenize the solution  
(160ml of DDN Surf in 8L of water (2%) for a fungicidal activity)
- ☒ **For the decontamination of radioactivity:**  
Dilute 500ml of DDN Surf in 5L (10%) of water and homogenize the solution

#### Use :

- 1- Clean the floor or surfaces or medical devices with a broom or a wipe or use the product by soaking
  - 2- Contact time: according to the antimicrobial activity desired (see disinfecting properties).
  - 3- Rinse thoroughly with water when using the DDN Surf by soaking.
- For the other applications, rinsing is not required *unless specified by the flooring manufacturer's instructions.*

### DISINFECTING PROPERTIES

Activity	Standards	Concentration	Contact time
<b>Bactericidal</b>	EN 13727 + A1	0,25%	5 minutes
	EN 1276 / EN 13697	1%	5 minutes
<b>Yeasticidal</b>	EN 13624	0,25%	5 minutes
	EN 1650	0,5%	5 minutes
	EN 13697	1%	5 minutes
<b>Fungicidal</b>	EN 13697	2%	10 minutes
	EN 1650	2%	15 minutes
<b>Virucidal</b>	EN 14476 PRV ( <i>hepatitis B model virus</i> )	0,25%	1 minute
	EN 14476 + A1 HIV	0,25%	15 minutes
	EN 14476 + A2 <i>Rotavirus</i>	0,25%	10 minutes
		0,5%	5 minutes
EN 14476 + A2 <i>Vaccinia virus</i>	0,25%	10 minutes	
	0,5%	5 minutes	
<b>Sporicidal</b>	EN 13704 <i>C.difficile</i>	1%	15 minutes

### PRECAUTIONS OF USE

See Safety Data Sheet

### FEATURES

- ☒ Aspect: **Liquid**
- ☒ Color: **Green**
- ☒ pH at 0.25%: **6.4**
- ☒ pH at 2%: **7.0**
- ☒ pH at 10%: **7.0**
- ☒ Expiration date: **2 years**

### COMPOSITION

- ☒ Non-ionic surfactants
- ☒ Sequestrants
- ☒ Quaternary ammonium
- ☒ Mint essential oil

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Hazardous product - Follow precautions of use. Use biocides cautiously. Always read label and product information before use.

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